

TECHNICAL DATA SHEET

PROTEIN TEST

TECHNICAL DATA SHEET

PRODUCT CODE	:	4APRO
RELATED STANDARDS	:	4A Medical PRO Test is under the scope of ISO 13485 and 9001 standards.
INTENDED USE	:	Pro test is used to control whether cleaning process is successful or not and if there is protein residue on the surface.
OPERATION CONDITIONS	:	It is necessary to wear gloves in case of contaminations due to user before startin analysis. The cotton bud which is present inside the test kit is rubbed onto the parts of the medical and surgical instruments that are suspected to have protein residue, in order to transfer the residue on to the cotton bud. If the residue is dry, it can be soaked with pure water and the process can be repeated. After, the cotton bud is put into the violet-capped tube and shaken for 5 seconds. After shaking, the tube is placed upside down on a flat floor. After 3 minutes, if there is violet-pink color on the bud or in the fluid, it shows that there is protein on the suspected medical/surgical instrument. Excessive amount of protein on the region that the test is applied may turn in the fluid completely violet. The color change after 3 minutes must not be taken into consider.
STORAGE AND DISPOSAL	:	Test kit must be stored in dark room without humidity at 4-20°C. Expiration date is due to 18 months after production date Each box contains 24 pieces.
PACKAGING	:	24 pcs in box.